

MEDICAL COMPRESSION STOCKINGS













Making life easier: this has always been the ultimate aim of VenoTrain compression stockings. As a medically proven form of treatment, these compression stockings counteract venous complaints, keeping the legs fit for the demands of everyday life.

Scientific studies show that one in five women and one in six men suffer typical symptoms such as varicose veins or heavy legs, due to a hereditary predisposition or strain at work, as well as the associated secondary risks.


VenoTrain compression stockings noticeably improve circulation and are available in various styles, compression classes and designs.

In standard cases, highly elastic compression stockings in compression class 1 or 2 are recommended. Since an optimal fit is crucial for the medical effect, your legs are individually measured for your stockings by your health care professional.



As an alternative to the custom-made solution, the Bauerfeind Perfect-Fit System allows for a high proportion of compression stockings to be provided from the standard product range.



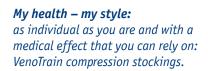

This brochure provides information about selected VenoTrain products for mild to moderately severe vein problems, as well as for medical preventive action and support during pregnancy.

For further information, please consult your health care professional or visit our website at www.bauerfeind.com





# AE







CREAM

HONEY

CARAMEL

AMARETTO

ALMOND

**ESPRESSO** 

BLACK

GRAPHITE

NAVY

Ccl1 | Ccl2



# **VenoTrain**<sup>®</sup> *micro*

The elegant classic: for everyone who feels young at heart.

The internationally popular compression stocking with a particularly high microfiber content for lightweight wearing comfort.

Available with an opaque finish in many designs and with a wide selection of discreet shades to suit your individual style.

The VenoTrain micro is also ideally suitable for work and travel, as well as for preventive care and during pregnancy.



"I am delighted that the VenoTrain micro supports me through my day!"

Birgit (housewife)





IVY

**BERRY** 

GRAPE



JAZZ

**ESPRESSO** 

**GRAPHITE** 

**BLACK** 

IVY

**BERRY** 

LIME

GRAPE

**SWING** 

**ESPRESSO** 

GRAPHITE

**BLACK** 

IVY

**BERRY** 

LIME

GRAPE

Ccl1 | Ccl2



The new design edition: dressed to impress with contemporary style.

well.

the modern compression stocking with a particularly high microfiber content:

with discreet shades or in the latest fashion colors for a confidently stylish appearance for both work and leisure.





"This won me over straightaway: a branded product with a modern, functional design – it suits me perfectly!"

Franziska (architect)



BOARDING PASS / EINST

CARAMEL

**BLACK** 

Ccl1 | Ccl2



# **VenoTrain**<sup>®</sup> *micro balance*

With an integral skin care substance: so you can feel comfortable in your own skin.

Sensitive skin needs the right care, yet many creams are unsuitable for use with modern compression stockings.

The VenoTrain micro balance is particularly skin-friendly thanks to its high microfiber content, and it has also been enriched with active skin-care substances so that your skin will remain smooth and supple, and the stockings are even easier to put on and take off.

The stockings come with an easy-to-use reload emulsion for skin care with a lasting effect that has been proven in clinical studies.



"A compression stocking that provides cream for my legs, keeping the skin supple: it just does me good!"

**Ulrike** (beautician)







Fashionably chic with a glossy effect: for everyone who loves something special.

A real highlight among medical compression stockings thanks to its brilliant shine:

figure-enhancing with a push-up effect, stretch comfort and special trilobal fiber for a perfect fit.

The fashionable accessory that is also suitable for preventive care and during pregnancy.



"With this combination, I always look good – even on special occasions!"

**Ulrike** (beautician)











CARAMEL

TOFFEE

**BLACK** 

GRAPHITE

Ccl1 | Ccl2







NATURAL CARAMEL

ESPRESS0

BLACK

NAVY

Ccl1 | Ccl2



# $\textbf{VenoTrain}^*soft$

Just naturally and wonderfully soft: comfort, just as I like it.

The VenoTrain soft makes life easy: extremely skin-friendly, soft and comfortable, robust and reliable, and for both him and her.

Available in many designs with a proven medical effect and a beneficial massage action.

Also available as the VenoTrain soft S, featuring a wide heel and comfortable instep for people who spend much of their time sitting.



"It's lovely and soft and helps me get through my work. And I'm always glad to have a bit of help!"

**Bettina** (gardener)





# **VenoTrain**® business

Perfect for a long day – while traveling or at work.

Timelessly elegant with a discreet ribbed appearance, the VenoTrain business meets every requirement for a high-quality men's stocking:

the special mix of cotton and Tactel® sports fiber offers first-class wearing comfort with thermal regulation.

Highly elastic LYCRA® soft and the extrawide comfort top ensure that the stockings are easy to put on and stay perfectly in position.

With a reinforced sole and Timbrelle® hightech fiber for long-lasting freshness and healthy foot conditions.









TERRA
BLACK
GRAPHITE
NAVY

Ccl1 | Ccl2





# **VenoTrain**<sup>®</sup> *glider*

VenoTrain® glider is the practical aid for putting on and taking off all types of medical compression stockings.

#### Hard-wearing material.

Stable and tear-resistant, extremely lightweight.

### Time- and space-saving.

Quick and easy donning and removal of products, also ideal for travel.











### The correct care – and safe handling.

Compression stockings are individually fitted and should be worn daily.

We recommend washing them every day (by hand or in a 30 degree machine wash) with a suitable detergent in order to ensure the lasting secure grip and elasticity of your stockings. When you receive your first pair of compression stockings, consider purchasing a second one so that you can still wear your stockings when one pair is in the wash.

Compression stockings should be put on gradually. Before putting them on, your legs must be dry and free from any creams, oils or ointments. When putting on your stockings, please do not over-stretch them or pull them up by the top band; instead spread the material smoothly and evenly over the leg. Please take particular care if you have sharp finger nails or are wearing jewelry.

Practical accessories such as Bauerfeind gloves and the VenoTrain glider make your compression stockings particularly easy to handle.

## **VenoTrain**<sup>®</sup> Accessories



### **Protects your stockings.**

A **lingerie bag** is a safe way to protect VenoTrain® compression stockings with every machine wash.



### Gentle on your stockings.

The **special detergent** cleans gently, ensuring careful treatment of and providing lasting protection for your products.



### Helps your stockings stay in place perfectly.

The **silicone top band cleaning cloth** is used to make sure that the top band keeps the stockings in place even on long days.



### Making it easier for you.

**Rubber gloves** make donning compression stockings easy.



### Wears well all day long.

The **non-skid sole** prevents slipping out.



#### MEDICAL COMPRESSION STOCKINGS

SUPPORTS AND ORTHOSES

INSOLES AND SHOES

MEASUREMENT TECHNOLOGY

#### Australia

MAS Medical Pty Ltd 44 Highbury Road Burwood, VIC 3125 Phone 03 9830 6529 Fax 03 9830 6517

E-mail info@masmedical.com.au

#### Galien

1200 Bergar Laval, Quebec, Canada H7L 5A2 Office Local (450) 629-9889 Office Toll Free 1-877-629-9889 Fax Local (450) 629-9895 Fax Toll Free 1-877-629-9895 www.galiengroup.com

#### Singapore

Blk 41 Cambridge Road #01-21 Singapore 210041 Phone +656396 3497 Fax +656295 5062

E-mail info@bauerfeind.com.sq

Bauerfeind Singapore Pte Ltd.

United Kingdom Bauerfeind UK

Phyllis House
229 Bristol Road
GB-Birmingham B5 7UB
Phone +44(0) 121 446 53 53
Fax +44(0) 121 446 54 54
E-mail info@bauerfeind.co.uk

#### USA

Bauerfeind USA, Inc. 3005 Chastain Meadows Parkway Suite 700 Marietta, GA 30066 Phone 1800 423 3405 Phone (770) 429 8330 Fax (770) 429 8477

E-mail info@bauerfeindusa.com

Rev. 1 - 05/12\_27120102051001

#### **Bauerfeind AG**

Triebeser Strasse 16 07937 Zeulenroda-Triebes Germany

Phone +49 (0) 36628-66-10 00 Fax +49 (0) 36628-66-19 99 E-mail info@bauerfeind.com